FR URGO SCARFORM / URGO SCARFORM PRO-S 7g

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GB URGO SCARFORM / URGO SCARFORM PRO-S 7g

URGO SCARFORM / URGO SCARFORM PRO-S is a silicone gel with a skin tone effect, composed of a medical-grade silicones. These components were selected for their biocompatibility. URGO SCARFORM / URGO SCARFORM PRO-S is used in the treatment of hypertrophic and keloid scars, and in aesthetic enhancement: it visibly reduces your scars, whether they are recent or old.

URGO SCARFORM / URGO SCARFORM PRO-S has the following beneficial effects:

- Prevention
- Healing

URGO SCARFORM / URGO SCARFORM PRO-S is mainly used on the face and in bending zone areas and on body joints where silicone dressing is not easy to apply. After application URGO SCARFORM / URGO SCARFORM PRO-S forms only a protective and invisible film on the skin.

USES

The intrinsic qualities of the silicone gel URGO SCARFORM / URGO SCARFORM PRO-S improve appearance of old and new scars.

- They cure hypertrophic or keloid scars, whether recent or old.
- They prevent closed wounds from becoming keloid or hypertrophic scars (following surgery, c-section or burn injuries).
- URGO SCARFORM / URGO SCARFORM PRO-S can be used on children's scars.

CONTRAINDICATIONS

URGO SCARFORM / URGO SCARFORM PRO-S cannot be used in the following cases:

- Open wound (following surgery or burn injuries),
 - Maceration,
 - Mucosa and close to eyes,
 - Do not inhale or ingest URGO SCARFORM / URGO SCARFORM PRO-S.
- In event of skin reaction, suspend treatment and consult your doctor.

SECONDARY EFFECTS

No known side effects.

DIRECTIONS

- Before use, clean and dry the scar thoroughly (using a neutral soap).
- Apply a small quantity of URGO SCARFORM / URGO SCARFORM PRO-S directly to the scarred area. Allow to dry for a few seconds.
- URGO SCARFORM / URGO SCARFORM PRO-S must be applied twice daily – morning and afternoon.
- Put the top back on the tube carefully after use.

It is recommended to continue the treatment for 2 to 4 months minimum. Treatment can be prolonged for up to 1 year.

With friction, a part of the silicone protective film can be removed. As long as a fine amount of the silicone film remains on the skin, leaving on your skin with a silky feel.

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