


INFECTIOUS DISEASES

Bioline™ EV71 IgM

IgM ANTIBODIES TO ENTEROVIRUS 71 TEST

Bioline™ EV71 IgM test is a rapid, qualitative and differential detection of IgM antibodies to Enterovirus 71 in human serum or plasma.

- Early diagnosis of acute EV 71 infection
- Differential detection of IgM antibody
- Easy to use: No need of any equipment
- Test result: 15-20 minutes
- Specimen: serum, plasma (5 µl)
- Shelf life and storage temperature: 18 months from the date of manufacturing at 1-30 °C

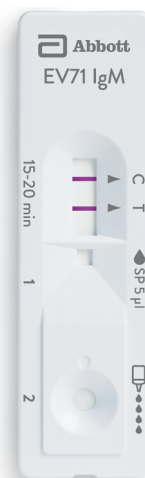


MATERIALS PROVIDED

- Test device
- Assay diluent
- Capillary pipette (5 µl)

SIMPLE PROCEDURE


1 Add Specimen

Dispense 5 µl of serum or plasma into the specimen well "1".

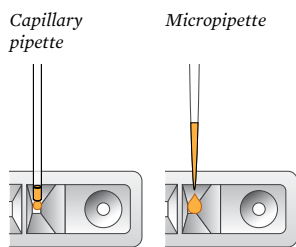


2 Add Assay Diluent

Dispense 3-4 drops of the assay diluent.

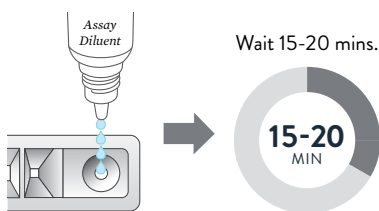

RESULTS INTERPRETATION

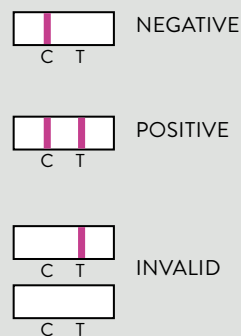

The presence of any test line, no matter how faint, the result is considered positive.

ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
EV71 IgM	43FK50	Device	Serum/Plasma	25T/Kit