


Bioline™ NOROVIRUS

NOROVIRUS ANTIGEN TEST

The Bioline™ Norovirus test is a chromatographic immunoassay for qualitative detection of the presence of norovirus antigen (Genogroup I (GI) and Genogroup II (GII)) in human fecal specimens. It is used as an aid in the diagnosis of acute gastroenteritis with the symptoms of suspected gastroenteritis caused by Norovirus.

- Easy to use
- Specimen : Fecal specimen (50-100 mg)
- Test result : 15 minutes
- Shelf life and storage temperature: 24 months from the date of manufacturing at 1-30 °C
- Performance: Sensitivity 84.1 %, Specificity 96.1 % (vs realtime RT-PCR)

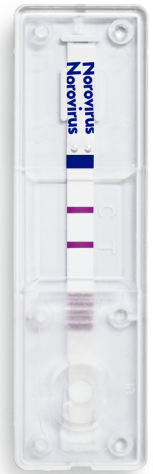

MATERIALS PROVIDED

- Test device
- Specimen collection tube
- Assay diluent
- Specimen collection swab
- Disposable dropper
- Disposable dropping cap

SIMPLE PROCEDURE

1 Add Assay Diluent
Transfer assay diluent twice.

2 Insert swab specimen
Insert the swab into the specimen collection tube and swirl the swab at least 10 times.

3 Assemble dropping cap
Assemble dropping cap on the specimen collection tube.

4 Add Specimen
Dispense 4 drops.

RESULTS INTERPRETATION

NEGATIVE

POSITIVE

INVALID

The presence of any test line, no matter how faint, the result is considered positive.

ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
Norovirus	52FK10	Device	Fecal	20T/Kit