190 c

Dispositive medical
Dispositif medical
Medical device

EKUBERG
pharma

MADE IN ITALY
Via Fosselle, 38 - 73025-Montone (LE)
Certified company ISO 9001:2008
www.dermosart.com
customer service: ekuberg@dermosart.com

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Nhà sản xuất:

MADE IN ITALY
Via Pozzello, 36 - 73025- Martano (LE)
Certified company ISO 9001:2008
www.dermoxen.com
customer service: ekuberg@dermaxen.it

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DermoXEN®

Dermoxen® BACTOR vaginal ovules

with protective, restoring and soothing action, are recommended for the treatment and prevention of **vaginitis and bacterial and fungal vaginitis**. Thanks to the presence of **Hyaluronic Acid** (sodium salt), they favour the regeneration of vaginal mucosa.

Suggested treatment:

- in **case of recurrent vaginitis and vaginosis**;
- to attenuate **itching, burning and fight bad odour**;
- to restore vaginal ecosystem, **even in case of diabetes and/or stress**;
- to prevent infections **after surgical procedures**.

Directions for use and dosage: wash your hands carefully, extract one single ovule and insert it deeply into the vagina, in a supine position, before going to bed. It is recommended to use 1 ovule a day for 7 days or according to your doctor's advice.

Composition: hyaluronic acid (sodium salt), Vitamin E, lipophilic extract of Aloe vera, Chlorhexidine dihydrochloride, lactic acid, semisynthetic glycerides.

Warnings: avoid contact with eyes; in case of contact, wash with plenty of water. Keep out of reach of children. Do not swallow. Store in a cool and dry place, away from heat sources (at temperature lower than 25°C). Do not use after the expiry date indicated on the box. Expiry date refers to the product properly stored. Do not use if the package is opened or damaged. Do not exceed recommended dose, unless otherwise advised by your doctor. In case of sensitization, stop using the product and consult your doctor. Do not use on damaged mucous membrane.

Rev. 0 del 11/2017

EKUBERG
pharma
MADE IN ITALY
Via Pozzella, 36 - 73025 - Marone (BR)
Certified company ISO 9001:2008
www.dermoxen.com
customer service: ekuberg@dermozen.it

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