## LUMIX 0.3 EYE DROPS

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### THÀNH PHẦN

Sodium Hyaluronate 0.3 %

### MÔ TẢ

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### Sản xuất bởi:

East Midlands Pharma Ltd

Unit 1, Crosslink 646, Rothley Lodge Commercial Park, Rothley, Leicestershire, LE7 7NL, United Kingdom.

## LUMIX 0.3 EYE DROPS

Eye drops

### INGREDIENTS:

Isotonic buffered solution with Sodium Hyaluronate 0.3% and PHMB

### USE:

Lumix 0.3 have been specially developed to provide effective and long lasting relief for dry, tired and irritated eyes

### WARNINGS:

Do not use if seal is broken. Do not touch your eye with the dropper tip to avoid scratching your eye and contaminating the solution inside the bottle. Do not use if allergic to any of the ingredients. If eye irritation occurs, discontinue use and consult your pharmacist or optician. Do not use at the same time as using other eye preparations. Do not swallow. Do not use after the expiry date. Use within 90 days of opening.

### DIRECTIONS:

Drops can be used without having to remove contact lenses. For use in eyes only.

- 1.Wash your hands before use.
- 2.To open, twist cap anti-clockwise.
- 3.Two drops per eye as often as needed.
- 4.Close bottle tightly after use.

### STORAGE

Store between 5°C to 30°C. Avoid direct sunlight

### Manufactured by :

East Midlands Pharma Ltd

Unit 1, Crosslink 646, Rothley Lodge Commercial Park, Rothley, Leicestershire, LE7 7NL, United Kingdom.