Smartbibi
NASAL

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ISO 9001 : 2015
ISO 13485 : 2016
GMP CFR21 FDA

CE 0477 15ml e
REPRODUCTION OF MEDICAL DEVICE

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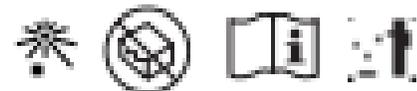
Fabbricante/Manufacturer
GRICAR CHEMICAL srl

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Smartbibi NASAL

MEDICAL DEVICE

Decongestant in the case of Rhinitis

SMARTBIBI NASAL is a nasal spray of hypertonic saline solution at 3%, with the addition of hyaluronic acid. SMARTBIBI NASAL has a decongestant action on the nasal mucosa in addition to a fluidising effect on the mucus. It helps the drainage of secretions and is therefore useful for keeping the nasal passages clean, especially in the case of cold, allergic rhinitis, rhinosinusitis and sinusitis. It is useful to improve the post-surgical course of the nasal cavities or the course of acute rhinosinusitis and nasopharyngitis.

HOW TO USE: Shake before use. At first use or after a prolonged period of inactivity, actuate the dispenser several times until the first product is dispensed. To activate the applicator, press with 2 fingers on the base of the dispenser.

- 1 nebulization per nostril up to three times a day for children from 3 to 12 years.
- 1 or 2 sprays per nostril up to four times a day up to 12 years old.

COMPOSITION: Water, Sodium hyaluronate, Sodium chloride, Thyme distilled water, Potassium Monobasic Phosphate, Potassium Bibasic Phosphate, Potassium sorbate.

WARNINGS: Avoid contact with eyes. Do not use if the package is open or damaged. There are currently no known interactions with drugs. In case of undesired reactions stop the treatment, consult the doctor and inform the Manufacturer. In case of hypersensitivity or individual allergy to one or more components, avoid using the product. If symptoms persist after 3-5 days consult a doctor. Keep out of reach of children. Do not dispose of the container after use. Use in children must be supervised by an adult. In case of pregnancy or lactation, take the product under medical supervision

STORAGE: Do not use after the expiry date indicated on the package; the expiry date refers to the unopened and properly stored package. Store at room temperature avoiding exposure to direct sources of light and heat and / or excessive environmental humidity; avoid contact with water. Once opened, use within 1 month.

NATURALLY LACTOSE FREE

GLUTEN FREE

15ml

FABBRICANTE / MANUFACTURED BY: Gricar Chemical S.r.l - Via San Giuseppe 18/20, Brugherio (MB) ITALY

 Fabbricante/Manufacturer
GRICAR CHEMICAL srl
Via San Giuseppe, 18/20
Brugherio (MB) - ITALY
e-mail: solulife@gricar.net

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