
BiOHealth[®]
Hi-Tech for Healthy life

**Air Compress Nebulizer
Instruction
BIOHEALTH NEB PRO**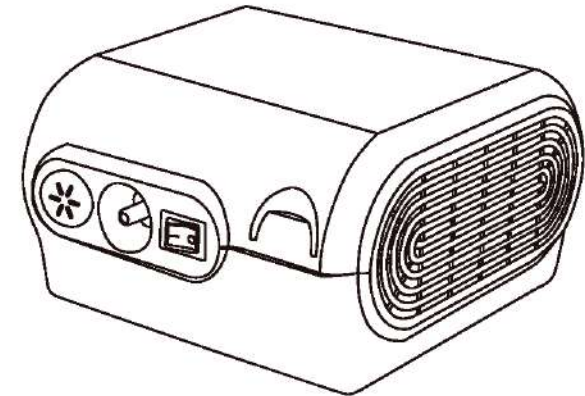

• NEB PRO

CONTENT

Chapter 1	Product Features	-----	1
Chapter 2	Usage and Maintenance	-----	2
Chapter 3	Others	-----	6

CHAPTER 1 Product Features

I Summary

Medical air compressing nebulizer is for atomizing drugs and inhalation therapy. It is a new generation of atomizer manufactures by our company and based on the improvement and innovation of similar products at home and abroad.

II Product Features and Indexes

1.Power supply:~220V± 220V 50Hz±1Hz

2.Max nebulizing rate:≥0.1mL/min

3.The maximum compression of compression pump:≥0.15MPa

4.The fuse tube T1.6AL250V.Φ5x20

5.Free air flux compressing pupm:≥10L/min

6.Noise:≤65dB(A)

7.Net weight: 1.9 kg

8.Dimensions: 17.0 x 18.2 x 10.3 cm

9.Input power:130VA

10.Normal working conditions:

Ambient temperature range:10°C~40°C Relative humidity range:30%~75%

Atmospheric pressure range:860hPa~1060hPa

Note: When the storage temperature is below 5°C, before using the device should be placed in the normal operating temperature of the environment for more than 4 hours.

This machine is not suitable for flammable and explosive gases occasions.

Electrical safety requirements: II class equipment, B-type applied part.

Working system: Intermittent operation - opening 30 minutes, stopping 30 minutes.

III Structural characteristic and working principle:

- ▷ Host using oil-free single-cylinder piston pump as the gas pressure:
- ▷ The design is classic, compact structure, small volume control, simple operation, easy to carry
- ▷ The working principle schematic diagram:

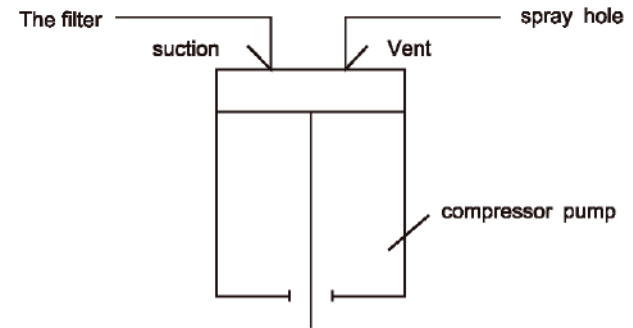
FIG.1:The working principle diagram

IV Electrical schematics

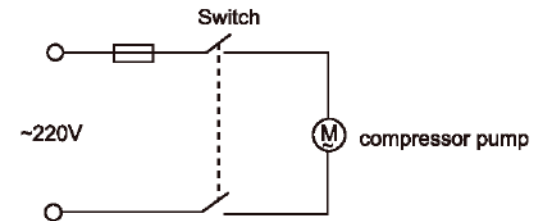
FIG.2:Electrical schematic diagram


Chapter 2: Usage and Maintenance

The use of the machine should be under the guidance of medical personnel who has professional qualifications, in strict accordance with the operating instructions. If in doubt, please contact the supplier or manufacturer.


I Unpacking and checking

Before installing and commissioning the product, user should check whether the product is in good working condition including the accessories in compliance with the list in the manual.

II Schematic diagram of the various components and the name of the machine

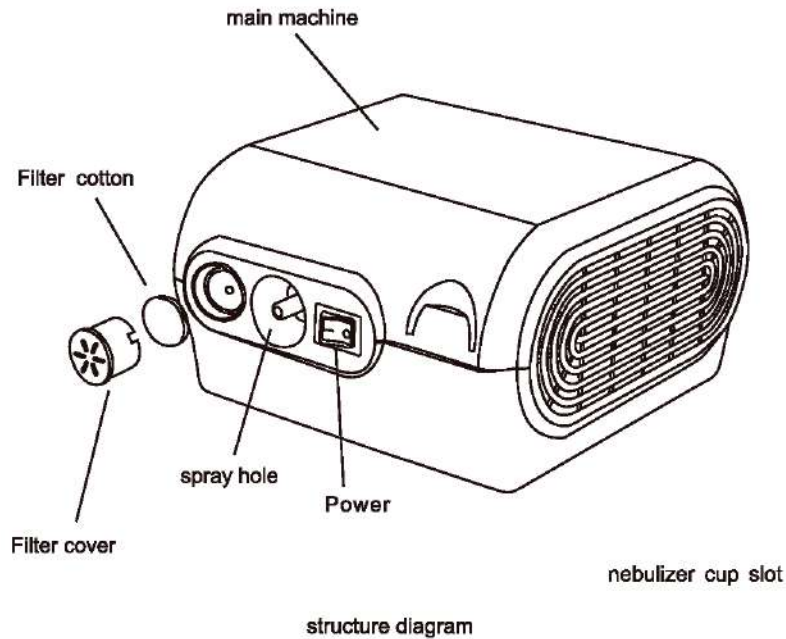Nebulizer structure diagram

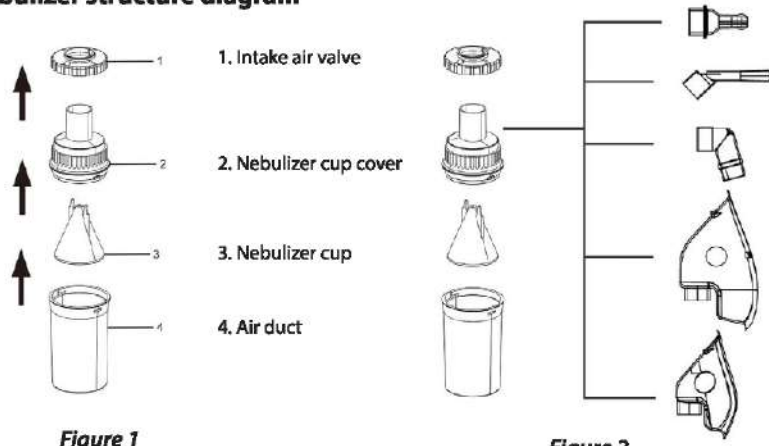PREPARATION

Open the nebulizer cup inlet flapper upwards (1) and according to the instructions from the atomizing cup (3) top Injection drug solution quantitative (see Figure 3). Liquid can not be injected over the atomizing cup (3) capacity scale online (maximum level 8ml)

The atomizing cup cover (2) vertically into the atomizing cup (3), note installation atomizing cup cover (2) of the 1.5 circular discs must be aligned with the mouthpiece (5) direction, then clockwise screwing.

Install intact valve (1) and a nozzle (5) (see Figure 4).

Use a mouthpiece (5) is the most effective way of inhalation therapy can be used this way directly to the drug enter the lungs, decrease the waster of medicine. Vertical atomization to hold the cup (3)

Gas conduct (4) the inhaler and atomization device connection.

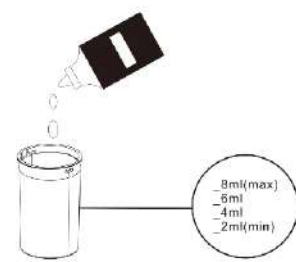
Figure 3

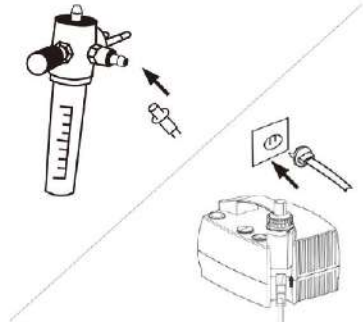
Figure 4

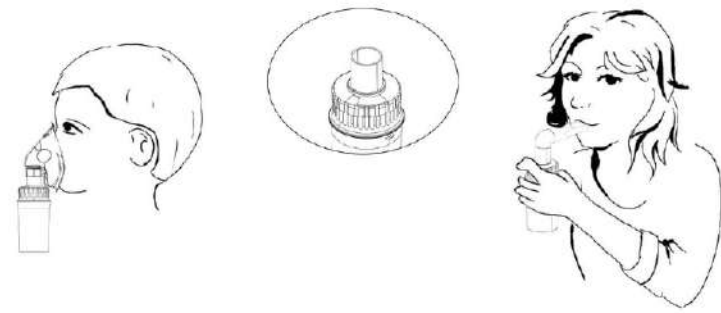
Figure 5

III The Method of Adding the Drug into the Sprayer and Installation

- ▶ Unscre the atomizing cup list
- ▶ Follow the doctor's advice, pour the drug moderately into the atomizing cup. Do not exceed the maximum scale line
- ▶ Screw the atomizing cup lid
- ▶ Connect the T-joint, bellow and mouthpiece.

⚠ Note: During the treatment with the mouthpiece into the oral cavity close your lips slightly.

IV Operating Steps

- ▶ Connect the host power supply
- ▶ Turn on the power switch
- ▶ After the host is working normally 1~2 minutes, to connect the connecting tube between the host mist outlet and atomizing cup
- ▶ Following the doctor's advice, execute an atomization treatment

V Proper Maintenance

- ▶ Upon shutdown, unplug the connecting tube and remove the mouthpiece
- ▶ Unscrew the atomizing cup cover, empty the residual liquid in the cup
- ▶ Make the connecting tube, mouthpiece, atomizing cup lid, spray baffle, atomizing cup, corrugated pipe, T-type joints, etc. Rinse with tap water or immerse in warm water for 15 minutes. In order to thoroughly wash add some vinegar to the water.

⚠ Note: Do not use boiling water to clean the product to prevent thermal deformation.

- ① After cleaning all the accessories, dry them then store.

VI Replacement of filter cotton

- ▶ In general, the filter cotton must be replaced every three (3) months. If the cotton is found to be wet, replace it. The cotton to be used should be dry.
- ▶ When replacing the cotton, pull out the filter cover from the host first.

VII Replacement of Fuse Tube

- ▶ open the case, take and unscrew the tube socket, then replace.

⚠ Note: Unplug the power plug prior to opening the case to cut the power supply.

VIII Other Reminders

- ① Do not fold the connecting pipe during storage
- ① When not in use for a long time, disconnect the power plug
- ① Storage must be out of reach by children
- ① Fragile to transport and storage

Ambient Temperature Range: -40°C~+55°C Range of relative temperature : 10%~93%

Atmospheric pressure range: 700hPa~1060hPa

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
⚠ Note: The nebulizer should be stored in a non-corrosive gas and well ventilated room. Avoid agitation when transported.

- ▶ Warning:

For safety usage of the product take note of the following:

Legend	Content
⚠Note	Erroneous usage can cause harm or injury to the user and product damage.
①	Take note of important matters. It is crucial to understand the meaning of the pictures or images shown to the left is for general information.
⊘	The image to the left means general prohibitions.

IX Symbols and their meanings relevant to safety measures in the machine

SYMBOLS	THE MEANING OF SYMBOLS	SYMBOL	THE MEANING OF SYMBOLS
	Alternating Current (AC)		Type B Applied Part
	Class II Apparatus		Direct to the general power supply
	Power Off		This side up
	Fragile		Keep dry
	Note: Refer to the Manual		

X Trouble shooting

Serial Number	Product Glitches	Possible Cause	What to do	Remarks
1	Loud sound during usage	Filter cotton is not properly loaded	Mount the filter cotton	
2	Machine is not working	Power is not switched on	Make sure that the power is turned on and the plug is properly inserted to the power supply socket.	
		Fuse	Fuse is lapsed	Replace the fuse tube
3	machine working but sprayer not spray	Clean the spray or it may be deformed.	Change the atomizer	When used for 300 times it should be replaced
		Spray the cleaning Has been deformed disinfection		
		The atomizing cup cover is not installed correctly	According to spray the correct installation methods in place	
		Run out of liquid medicine	To join the amount of liquid medicine	
4	There are water droplets in the connection	Liquid adding excessive or cleaning after the connecting pipe is not dry	The connection pipe is connected to the host computer, boot, fingers opened and closed repeatedly connecting pipe port, to the exclusion of water	
Additional questions, please contact the supplier or manufacturer, or please professional repair				

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