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sodilax ENTEROCOLISMA Medical Device

Enema Sodilax 120 ml

Composizione:
Each bottle contains sodium phosphate monobasic dihydrate, disodium phosphate dodecahydrate, sodium methyl p-oxylbenzoate, purified water.

Mechanism of action:
Moves the stool and facilitates its evacuation.

Indications:
Constipation, colon cleansing, preparation for colonoscopy of sigma, rectum, X-ray, surgery, colon surgery.

Contraindications:
The enema should not be used in case of chronic abdominal pain of unknown origin.

Side effects:
May cause abdominal pain and diarrhea in sensitive patients.

Dosage:
Adults and teenagers (12-18 years old): 1 enema of 120 ml. Do not use more than one enema per day.

Instructions for use:
The packaging consists of one bottle and one cannula.
 Acid Sodium Phosphate Enema can be used at the room temperature. If preferred warm, just put in contact with hot water (by immersion or under the faucet).
 For best results, lie down on the left side. (Fig. 2)
 Remove the protective bag from the rectal cannula, take off the cap from the bottle and put on the cannula.
 Take the bottle and gently insert the cannula in the rectum, squeeze the bottle till the bottom. When the administration is finished, extract the cannula and stay lying down until you feel an urgent stirring to evacuate, which usually arrives in 3 minutes.

Storage:
The product should be stored in a dry place at a temperature between 4° and 30° C. KEEP OUT OF THE REACH OF CHILDREN.

 Warning, see the instructions for use.

 Store between 4° and 30° C.

 Expiry date.

 Batch number.

 Do not dispose in the environment after use.

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New Fa Dem, S.r.l.
Viale Ferraresi delle Storie Zona AG,