Information leaflet:

STARBALM® COLD GEL
STARBALM® COLD SPRAY
STARBALM® INSTANT COLD PACKS

Read the information leaflet carefully before use.

STARBALM® COLD is the blue line of STARBALM® that focuses on cooling to prevent swelling and sports injuries.

STARBALM® COLD GEL

STARBALM® Cold Gel is a medical device. STARBALM® Cold Gel can be used during and after exercises or other physical activities. It helps the healing process of stressed and sensitive muscles and the healing of sports injuries.

Usage

STARBALM® Cold Gel can be applied more than once a day. The gel must be smeared out. The cooling sensation can be noticed immediately.

STARBALM® COLD SPRAY

STARBALM® Cold Spray is a medical device. The cold spray can be used for a fast cooling effect for sports injuries.

Usage

Shake well before use. Spray 15 cm from the skin. Prevent inhalation and contact with the eyes.

STARBALM® INSTANT COLD PACKS

STARBALM® Instant Cold Pack is an Icepack and is a medical device. Instant Cold Packs are perfect for use of injuries and to treat swelling.

STARBALM® Instant Cold Packs don't need to be kept cold, which makes them perfect to keep in your sportsbag! Perfect for application on acute injuries, where cold therapy is recommended.

Usage

STARBALM® Instant Cold Packs are activated by squeezing firmly. After squeezing the user shakes the pack, which will turn cold instantly. The pack will remain cold for 15 to 20 minutes.

Warnings

For external use only and apply on intact skin. May cause an allergic reaction in some individuals with sensitive skins; test on a small area first before use. Do not use in case of hypersensitivity to any of the ingredients. Immediately stop using if the skin becomes irritated or an allergic reaction occurs. If symptoms persist, consult your doctor. After application of the products, do not cover the skin with occlusive compresses. Do not use around the eyes, mucous membranes, wounds, sensitive areas and/or irritated skin. Not recommended for use by children under 6 years of age.

Storage

Keep out of reach of children.
Store in a cool and dry place.
Keep the spray away from sun and temperatures exceeding 50°C.

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