Please read instruction before use
ENG: HOW TO USE A CONDOM

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1. Put the condom on when the penis is erect, before there is any contact between the penis and your partner's body. Fluid released from the penis during the early stages of an erection can contain sperm and organisms that can cause STI's.

2. Condom should be opened at the jagged edge of the condom foil with care. (*) otherwise, it may be damaged.

3. Do NOT flip the condom inside out when use. Put the condom right on top of your glans penis. If there is air inside the condom, squeeze the teat to make sure there is no void space inside the condom.

4. Use your fingers to hold the top part of the condom and unroll the condom down the full length of the penis with another hand. If the condom is put on inside out, it will be unable to unroll. In case the condom has been put on from the wrong side, please dispose it and use a brand new condom.

5. Soon after ejaculation, withdraw the penis from your partner while it is still erect by holding the condom firmly in place. Remove the condom only when the penis is fully withdrawn.

6. Do NOT flush the used condom down the toilet, use tissue to wrap it and dispose in a rubbish bin.

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Please read this instruction before use

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