



**FARMACTIVE®**  
**SILVER**

## **Spray with colloidal silver and hyaluronic acid sodic salt**

125 ml spray bottle containing 10 g of powder

### **Destination of use**

Topic and temporary treatment of not infected skin lesions (e.g. cuts, abrasions, wounds, minor burns that did not damage derma, postoperative wounds, decubitus pressure ulcers, incontinence and diapers irritations).

### **Product description**

Farmactive Silver spray is a powdered spray device containing parts of colloidal silver and hyaluronic acid sodic salt. The colloidal silver has antibacterial properties helping to prevent the microbic contamination of the dressing and keep wound environment clear from external bacteria.

Hyaluronic acid is a polysaccharid naturally contained into connecting tissue of the human body where it has moisturizing and lubricating functions. During the healing process of a lesion, the moisturizing properties of hyaluronic acid helps the physiologic cells motion and the diffusion of the molecules soluble in water.

Silver and hyaluronic acid are in formulation with light kaolin and silicon dioxide that have absorbant properties and control the exudates, creating a favourable micro environment favourable to the natural healing of the wound.

### **Composition**

Colloidal silver 2%, hyaluronic acid sodic salt 0.2%, light kaoline, silicon dioxide, mixture of propellants.

### **User's instructions**

Carefully clean the wound, shake well the bottle and keeping it in vertical position about 10 cm far from the wound, spray a quantity enough to cover the surface of the wound. Spray in short intervals to measure out the quantity of the product and avoid the dispersion of the powder in the air. After the use, cover the zone with a sterile gauze or a sterile strip. Repeat the application once or twice a day, according to the needs, after checking the condition of the lesion and a correct cleaning of the part to treat.

In case of temporary jam of the bottle, shake strongly, or remove the plastic valve and wash it under lukewarm water. Dry the valve and reapply on the output nozzle.

### **Precautions**

The product is for external use, topic and temporary local use. It is recommended to not